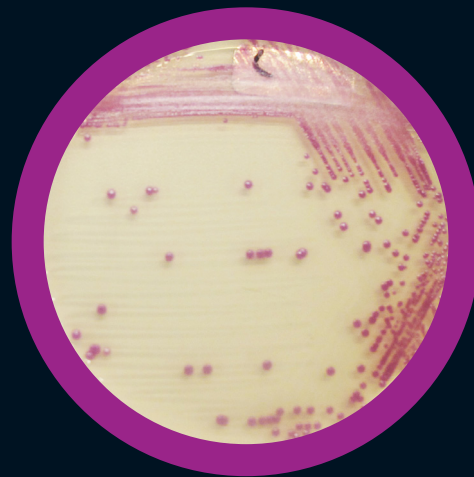
# LIM RambaQUICK™ StrepB Method

+

Broth enrichment  
LIM RambaQUICK™ StrepB

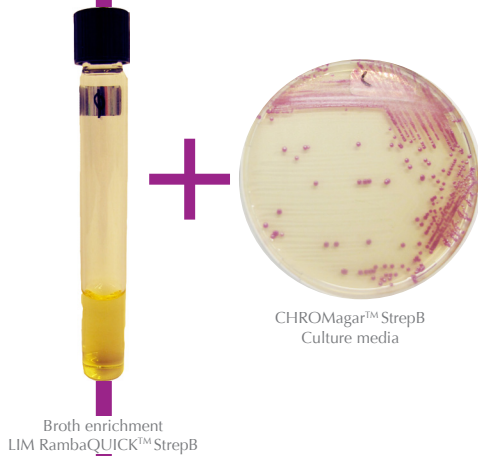
CHROMagar™ StrepB  
Culture media

For rapid detection  
of *Streptococcus agalactiae* from swabs

# LIM RambaQUICK™ StrepB Method

www.CHROMagar.com

For Rapid detection of *Streptococcus agalactiae* from swabs.


## Background

The CDC recommends pregnant women to be tested at 35 to 37 weeks of pregnancy for *Group B Streptococcal (GBS)* carriage by **selective enrichment broth of vaginal/anorectal swab samples**, followed by subculture. However, the selective broth formulas are not dedicated to the inhibition of *Enterococci*, usually part of the anorectal flora and able to hide the growth of GBS, source of false negative results.

LIM RambaQUICK™ StrepB is a selective enrichment broth **designed to largely reduce the growth of *Enterococci***, while allowing growth of GBS, and, therefore to raise the sensitivity of the prepartum screening method.
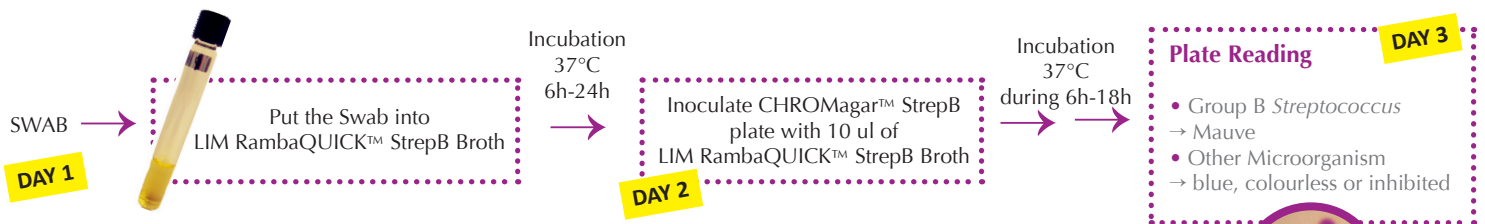
## LIM RambaQUICK™ StrepB Method

## Performance of the LIM RambaQUICK™ StrepB broth

Compared to classical LIM + subculture method:

### INHIBITION OF MOST ENTEROCOCCI

LIM RambaQUICK™ StrepB is a selective enrichment broth **designed to largely reduce the growth of *Enterococci***, while allowing growth of GBS, and, therefore to raise the sensitivity of the prepartum screening method.

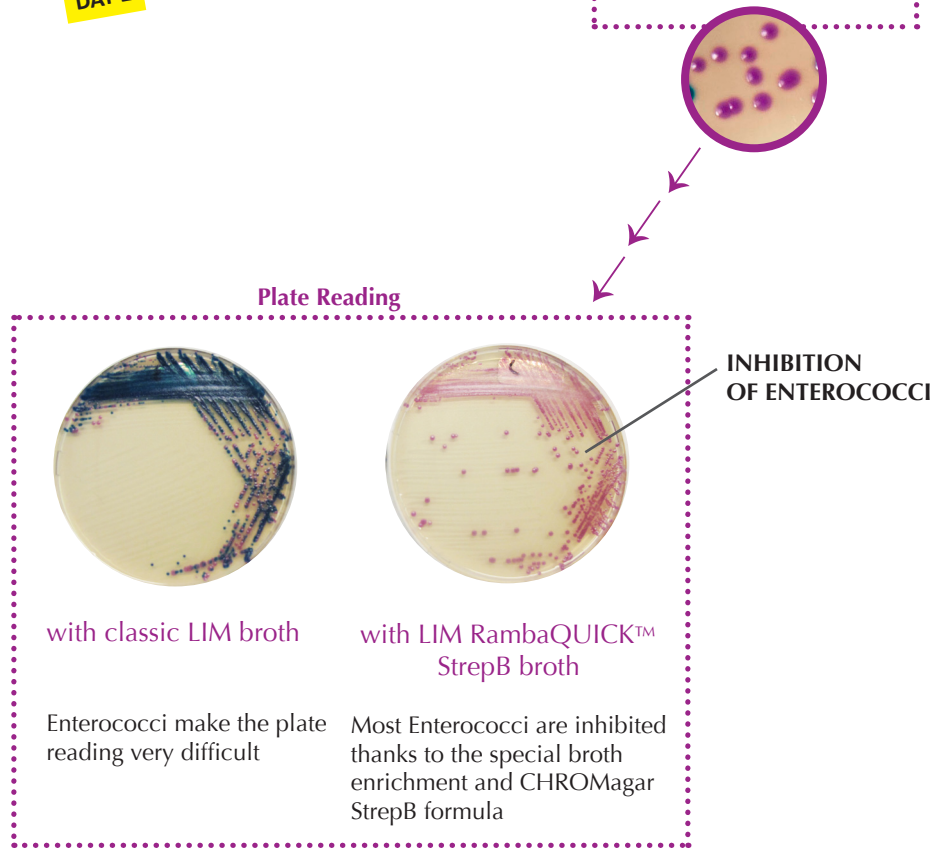
### BROTH INOCULATION

Classical Method recommends overnight broth incubation for classical LIM. LIM RambaQUICK™ StrepB can be used also with overnight incubation, but has an optimal differentiation at 6h only as suggested in the Dr Poisson Method Study.

Study: Prepartum vaginal/anorectal GroupB Streptococcus screening: Improvement of the enrichment step by the broth of additive RambaQUICK StrepB; D. Poisson et al. Centre Hospitalier Régional Orleans, France; Journal of Microbiology Methods 89 (2012) 107-109 doi:10.1016/j.mimet.2012.02.012

### EASY INTERPRETATION OF THE SUBCULTURE PLATE

Easier reading of CHROMagar™ StrepB plate thanks to an intense mauve colony colouration.

CHROMagar™ and RambaQUICK™ are trademarks of Dr A. Rambach - Pregnant women Picture rights: iStock

Version 1 / Feb -15  
LF-EXT-048

## Order References

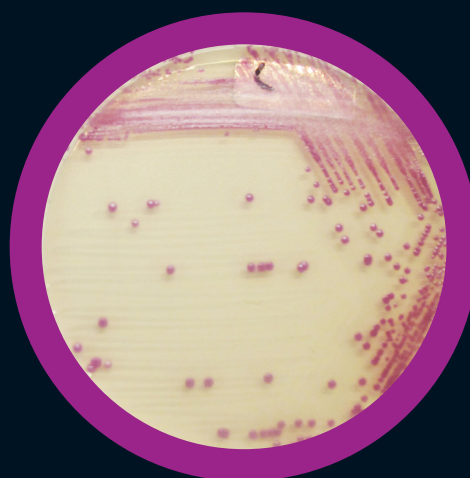
Please use these product references when contacting your local distributor:

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