





10-Minute ihcDirect® IHC Assays for IVD

ihcDirect is a novel 10-Minute immunohistochemistry (IHC) technology for intraoperative observation of important biomarkers on frozen tissues. ihcDirect tests are intended for *in vitro* diagnostic (IVD) use.

Assays are crafted by conjugating primary antibodies (abs) directly onto polymerized HRP enzymes, eliminating the need for secondary abs. This greatly enhances performance via simplified test protocols and faster turn-around times yet maintains the sensitivity of IHC assays. See our simplified frozen and FFPE tissue procedures!

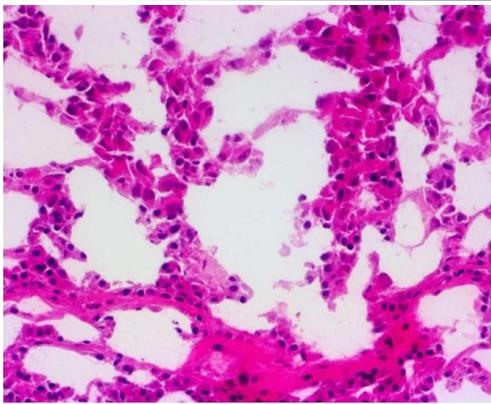

H&E Melanoma on frozen tissue

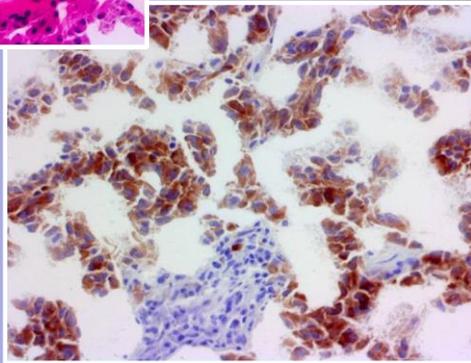

ihcDirect Mart-1 - same tissue block

ihcDirect Frozen Tissue Procedure	Time in mins.
Fixation:	Start
Wash (PBS-T)	
Blocking	1
ihcDirect pHRP Ab conjugate	3
Wash (PBS-T)	
DAB	1-3
Wash	
Counter Stain	20 sec
Rinse/Dehydrate/ Mount/Coverslip	User Det.
Total	<10

Simplicity of an H&E with the Sensitivity of an IHC...



ihcDirect[®] Antibodies and Reagents

ihcDirect assays registered for use as *in vitro* diagnostic tests are now available in two different formats: test kits with essential components included and now as separate components. To see what else is new, visit our updated website at www.Novodiox.com.

Antibody Test Kits & Antibodies and Ancillary Reagents

Monoclonal Antibodies (Anti-Human Clones)	Catalog Number	-005 (5ml)	-010 (10ml)	-015 (15ml)	-030 (30ml)
AE1/AE3 (Mouse AE1/AE3)	K31007-	Abs Only	Abs Only		
Calponin (Mouse, C126)	K41026-	Kit	Kit		
CEA (Mouse, C135)	K31038-	Abs Only	Abs Only		
CD45 (Mouse, C95)	K31010-	Abs Only	Abs Only	Abs Only	
CK5 (Rabbit, R226)	K43014-	Kit	Kit		
CK7 (Rabbit, R284)	K42015-	Kit	Kit		
CK 8/18 (Mouse, C94)	K41001-	Kit	Kit		
Pan-CK 4Abs (Mouse, Rabbit, AE1/AE3, C94, R226)	K41058- K31058-	Kit	Kit	Abs Only	
GFAP (Rabbit, R246)	K42002-	Kit	Kit		
Mart-1 (Mouse, A103)	K41009-	Kit	Kit		Kit
SMMHC (Mouse, SMMS-1)	K41024-	Kit	Kit		
Podoplanin (Mouse, C125)	K41025-	Kit	Kit		
Ancillary Reagents					
ihc DAB Kit	K50001-			x	x
ihc Blocker USA	K51002-			x	
Ihc Blocker CE Mark	K51001-			x	
ihc Wash Buffer Kit	K52001	800 ml			

Distributed by:

ImTec
DIAGNOSTICS NV

Contact: info@imtec.be | +32(0)3-2251139

IHC Tissue Tests/Kit Estimates

- 005 = 50 Tissues/bottle
- 010 = 100 Tissues/bottle
- 015 = 150 Tissue/bottle
- 030 = 300 Tissues/bottle
- @ ~100µl / tissue

1-510-342-3043
1-888-439-2716

www.novodiox.com

3517 Breakwater Ave.
Hayward, CA 94545